



NOTES :

MATERIAL : MOLDED OF 20% GLASSED FILLED ABS, AND SILVER/SILVER CHLORIDE COATED TO A MINIMUM OF 0.1 ML THICKNESS SPECIFICATION.

ALL DIAMETERS MUST BE CONCENTRIC TO WITHIN .003"



PROPRIETARY AND CONFIDENTIAL
THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF <INSERT COMPANY NAME HERE>. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF <INSERT COMPANY NAME HERE> IS PROHIBITED.

		UNLESS OTHERWISE SPECIFIED:	NAME	DATE		
		DIMENSIONS ARE IN INCHES	DRAWN	Crl	11.16.17	TITLE: 77P EYELET
		TOLERANCES: FRACTIONAL ±	CHECKED			
		ANGULAR: MACH ± BEND ±	ENG APPR.			
		TWO PLACE DECIMAL ±	MFG APPR.			
		THREE PLACE DECIMAL ±	Q.A.			
		INTERPRET GEOMETRIC TOLERANCING PER:	COMMENTS:			
		MATERIAL				
		FINISH				
NEXT ASSY	USED ON	APPLICATION	DO NOT SCALE DRAWING	SIZE A	DWG. NO. 77P EYELET-AGAGCL	REV
				SCALE: 2:1	WEIGHT:	SHEET 1 OF 1